



toll free: 1-877-706-4480
international/local: 713-706-4480
fax: 713-706-4161
www.MS3C.co

Does My Wheelchair fit into the MS3C-300WC Stair climbing Trolley for Wheelchairs?

1. Measure the distance between your wheelchair anti-tip bars, rear wheels and rear push bars. You will need at least 18" to clear the trolleys frame.

**You need AT LEAST 18" between the rear push handles,
Wheelchair wheels and anti-tip bars (if applicable)
to clear the trolleys frame**





toll free: 1-877-706-4480
international/local: 713-706-4480
fax: 713-706-4161
www.MS3C.co

2. Take the measurement from between your anti-tip bars, rear wheels and rear push bar and compare with the dimensions of the trolleys wheel supports below. Anywhere between 18" – 29" will work for the trolley.



**At Full in Position, Wheel Supports
extend 18" - 22"**



**At full out Position, Wheel Supports
extend from 22" - 29"**



toll free: 1-877-706-4480
international/local: 713-706-4480
fax: 713-706-4161
www.MS3C.co

3. Measure the wheelchairs backrest. You will need at least 18" to clear the frame. The height of the side supports should clear the wheelchairs rear push handles as well as the wheelchairs armrests.



Side Supports are height adjustable. This bar is from 18" to 27" when measuring from the ground. This will need to be adjusted if the side supports are hitting your wheelchairs armrests

Side Supports are Width Adjustable from 16" - 27"



4. Adjust the depth of the wheel supports if necessary.

Wheel Supports are depth adjustable to accommodate between 14" to standard Wheelchair Wheels

