

1. **Deploy Chair:** Pull lock knob located underneath seat to unfold chair.
2. **Raise Top Bar:** Push red horizontal bar to back of chair, simultaneously lifting top telescoping bar.
3. **Head Support:** Place and adjust head support on top telescoping bar.
4. **Secure Passenger:** Lock casters. Seat and secure passenger with 4-way lap/ shoulder safety belts.
5. **Push Handles:** Unlock casters. Lift push handles on back of chair and move into position at top of stairs.
6. **Guide Extension Handles:** Press down on red button and extend guide handles.
7. **Engage Tracks:** Push down on red foot pedal to release tracks.



8. Descending Stairs:

- a) Guide chair to edge of stairs.
- b) Fold down **Push Handles**.
- c) Pull back on extended **Top Bar** to slightly tilt chair backwards.
- d) One operator uses **Top Bar**, second operator uses bottom **Extension Handles** to guide chair tracks onto stairs.
- e) Proceed down stairs in safe, controlled manner.
- f) At bottom of stairs, raise chair back into upright position.
- g) Lift **Push Handles** and guide chair to safe location.



