



toll free: 1-877-706-4480
international/local: 713-706-4480
fax: 713-706-4161
www.MS3C.co

Folding Lightweight Ambulance Stretcher-Chair Item# MS3C-1000

Adjusting Backrest:

- Squeeze red lever located behind headrest and raise or lower backrest to desired position.

Raising and Lowering Siderails:

- To raise: pull up into the locked position
- To lower: pull release knob and lower into collapsed position

Loading Stretcher into Transportation Vehicle:

- Squeeze the red lever at the foot end of the stretcher.
- Push stretchers legs against loading platform, the legs will automatically collapse.

Unloading Stretcher from Transportation Vehicle:

- Pull stretcher out of vehicle, stretcher legs will automatically spring open into the outright positions and lock into place.

Adjusting Stretchers (5) Preset Positions:

- Push on the horizontal silver bar located at the head of the stretcher.
- Adjust up or down into the desired position.

YouTube Demo Video: <https://youtu.be/avuDDlimMZs>

