



toll free: 1-877-706-4480  
international/local: 713-706-4480  
fax: 713-706-4161  
[www.MS3C.co](http://www.MS3C.co)

## MS3C-250 Multi-Height Aluminum Alloy EMS Emergency Stretcher User Manual

### **Basic Operations:**

- The unit is designed for a one-person operation when loading into a transportation vehicle.
- For safety considerations, the unit comes with drop down side rails and 2 patient safety restraining straps.
- Two lockable swivel wheels, stretcher can be easily maneuvered and securely positioned next to a bed or a chair for an easy patient transfer.



### **Raising and Lowering Stretcher:**

- This unit is height and position adjustable (Trendelenburg and reverse Trendelenburg). To adjust the height of the stretcher, both operators will squeeze red lever located at both ends and slowly raise/lower the unit to any of the preset heights.
- To raise the stretcher, lift the stretcher until the legs spring open and lock into place.
- To position the unit into either Trendelenburg or reverse Trendelenburg, operator will squeeze red lever on the desired end and raise/lower. Each end works independently

### **Loading Stretcher into Emergency vehicle:**

- Only one operator is needed for patient loading.
- When loading stretcher into emergency vehicle, make sure the stretcher is in a level position.
- Push stretcher to the vehicle so that you can rest the safety wheels on the vehicle.
- Lift the foot end stretcher off the ground. Once the weight is off the wheels, squeeze the red release handle located at the foot end of the stretcher, push the stretcher into the transportation vehicle. The legs will automatically collapse. Secure the stretcher with the vehicle locking device.

### **Unloading the stretcher:**

- Release the stretcher from the vehicle locking device.
- Grab main frame and squeeze control handle located in the undercarriage.
- Keeping the stretcher level, pull the stretcher slowly from the vehicle until it reaches the safety stop.
- Stretcher legs will automatically lower and lock into place

**You Tube Demo:** [https://youtu.be/zgVQQs8eJ\\_c](https://youtu.be/zgVQQs8eJ_c)