

# MS3C-250 PRO Commercial Multi Height Aluminum Alloy EMS Stretcher User Manual

## **Basic Operations:**

- The unit is designed for a one-person operation.
- For safety considerations, the unit comes with drop down side rails and 2 patient safety restraining straps. Lift up to raise, squeeze red handle to lower.
- Two lockable swivel wheels, stretcher can be easily maneuvered and securely positioned next to a bed or a chair for an easy patient transfer. Head end casters are also lockable with pin located at top of caster.



## **Raising and Lowering Stretcher:**

- This unit is height adjustable to Trendelenburg and reverse Trendelenburg. To adjust the height of the stretcher with a single operator squeeze red handle at head end or green handle at foot end, raise/lower the unit to any of the preset heights. Same adjustments can be done simultaneously with 2 operators and corresponding handles.
- To raise the stretcher from collapsed position, squeeze corresponding handle (red at head end or green at foot end), lift the stretcher until the legs spring open and lock into place. This can be done with a single operator one end at a time or with 2 operators simultaneously.

## **Raise and Lower Backrest and Leg Rest:**

- To raise the backrest, squeeze red handle behind backrest, raise or lower to desired height
- To raise leg rest, lift to upright position, automatically locks into place. To lower, rotate red levers forward and lower leg rest to closed position

## **Loading Stretcher into Emergency vehicle:**

- Only one operator is needed for patient loading into a transport vehicle.
- When loading stretcher into emergency vehicle, make sure the stretcher is in a full upright, level position.
- Push stretcher to the vehicle and rest the head end loading wheels on the vehicle loading platform (ie: bumper).
- Lift the foot end stretcher off the ground. Once the weight is off the wheels, squeeze the green release handle located at the foot end of the stretcher, push the stretcher forward into the transportation vehicle. The legs will automatically collapse. Secure the stretcher with the vehicle locking device.

## **To remove the Backboard:**

- 2 Operators will be needed. Pull the red lever at the foot end of the stretcher and slide the backboard off of the stretchers frame.

## **Unloading the stretcher:**

- Release the stretcher from the vehicle locking device by grabbing main frame and squeezing lock release on locking mechanism. Keep the stretcher level, pull the stretcher slowly from the vehicle legs will automatically open and lock into place



Toll Free 1.877.706.4480 | Local/International 713.706.4480 | Fax 713.706.4161

[www.MS3C.co](http://www.MS3C.co)